

ANA 2000[®]

High Copper Admixed Alloy

EPOQUE 2000[®]

High Copper Admixed Alloy

N.B.
New mixing
times!

MIXING TIMES

POWDER	
1-spill	2-3 spill
30 sec.	35 sec.

CAPSULES		
Mixer r/min	No. 1	No. 2 & No. 3
3.500	10 sec.	12 sec.
4.000	8 sec.	10 sec.
4.500	7 sec.	9 sec.

The above values are valid for mixers such as Silamat S5, Ultramat 2, Espe Capmix and ProMix.

DUETT		
Mixer r/min	Duett 1+1	Duett 2+2
3.500	13 sec.	14 sec.
4.000	8 sec.	10 sec.
4.500	7 sec.	9 sec.

The above values are valid for mixers such as Silamat S5, Ultramat 2, Espe Capmix and ProMix.